THUASNE

Malleo Dynastab®

fr	Attelle stabilisatrice de cheville	5
en	Stabilising ankle brace	6
de	Orthese zur Stabilisierung des Sprunggelenks	8
nl	Stabilisatiebrace voor de enkel	9
it	Tutore stabilizzatore per caviglia	11
es	Férula estabilizadora de tobillo	13
pt	Tala de estabilizadora do tornozelo	14
da	Stabiliserende ankelskinne	16
fi	Stabiloiva nilkkatuki	17
sv	Stabiliseringskena för ankeln	19
el	Νάρθηκας σταθεροποίησης ποδοκνημικής	20
cs	Stabilizační ortéza kotníku	22
pl	Orteza do stabilizacji kostki	23
lv	Potītes stabilizēšanas saite	25
lt	Stabilizuojantis kulkšnies įtvaras	27
et	Hüppeliigese fikseerimise lahas	28
sl	Opornica za stabilizacijo gležnja	30
sk	Stabilizačná členková dlaha	31
hu	Stabilizáló bokarögzítő	32
bg	Стабилизираща шина за глезен	34
ro	Atelă stabilizatoare pentru gleznă	36
ru	Стабилизирующий ортез для лодыжки	37
hr	Udlaga za stabilizaciju gležnja	39
zh	踝部穩定夾板	40
ar	جبيرة تثبيت الكاحل	43

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

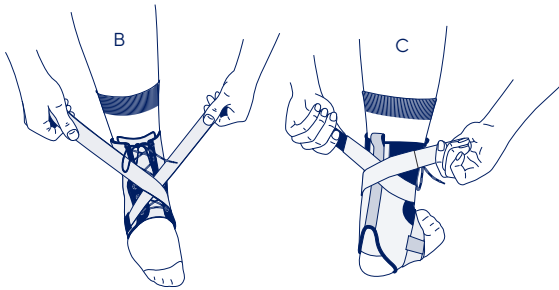
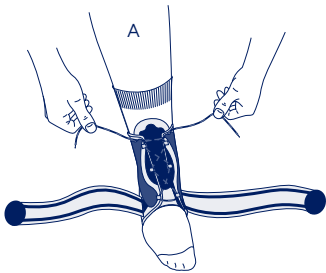
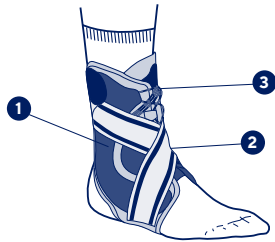
	1	19 - 22 cm	7 ^{1/2} - 8 ^{3/4} "
	2	22 - 26 cm	8 ^{3/4} - 10 ^{1/4} "
	3	26 - 30 cm	10 ^{1/4} - 11 ^{3/4} "

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	脚踝尺寸
ar	محيط الكاحل

fr	Immobilisation	Stabilisation	Proprioception
en	Immobilization	Stabilization	Proprioception
de	Ruhigstellung	Stabilisierung	Propriozeption
nl	Immobilisatie	Stabilisatie	Proprioceptie
it	Immobilizzazione	Stabilizzazione	Propriocezione
es	Inmovilización	Estabilización	Propiocepción
pt	Imobilização	Estabilização	Proprioceção
da	Støtte	Stabilisering	Proprioception
fi	Immobilisoi	Stabilisoi	Proprioseptio
sv	Immobilisering	Stabilisering	Proprioception
el	Ακινητοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
cs	Znehybnění	Stabilizace	Propriocepce
pl	Unieruchomienie	Stabilizacja	Propriocepcja
lv	Imobilizācija	Stabilizēšana	Propriocepcija
lt	Imobilizavimas	Stabilizavimas	Propriocepcija
et	Liikumatuks muutmine	Stabiliseerimine	Proprioceptsiioon
sl	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
sk	Imobilizácia	Stabilizácia	Propriocepcia
hu	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
bg	Обездвижаване	Стабилизация	Проприоцепция
ro	Imobilizare	Stabilizare	Propriocepție
ru	Иммобилизация	Стабилизация	Проприоцепция
hr	Imobilizacija	Stabilizacija	Propriocepcija
zh	固定	稳定	本体感受
ar	التثبيت	الاستقرار	استقبال الحس العميق

fr

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STABILISING ANKLE BRACE

Description/Destination

This device is an ankle brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: polyamide, polyester, neoprene, elastane.

Rigid components: stainless steel.

Properties/Mode of action

Rigid side reinforcements to ensure ankle stability. ①

Straps that optimise the action of the reinforcements. ②

Fast, easy lace fastening. ③

Bilateral product.

Indications

Moderate to severe ankle sprain.

Chronic ankle instability.

Ankle pain/oedema.

Sport activities resumption.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product in a medical imaging machine.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device. It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral stays. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip the brace over the foot.

Pull the laces to close the brace, without tightening excessively to avoid any tourniquet effect. (A)

Cross the straps over the top of the foot once, then cross them a second time at the base of calf before attaching the self-fastening tabs to the sides of the ankle brace. (B-C)

In the event of discomfort in the ankle bone area, the position of the frame can be modified, by sliding it above or below the stitch located at the bottom of the frame cover.

Care/maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Make sure to remove the lateral stays before washing.

Replace in the original location before next use. Machine washable at 40 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not

tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2043601 (2023-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros