
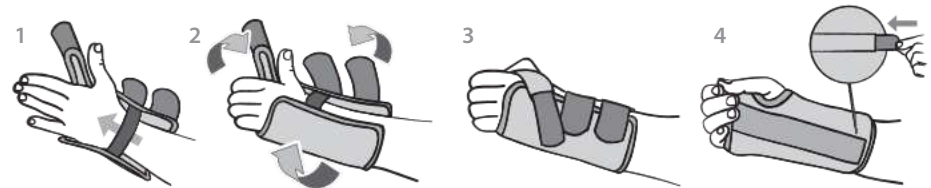
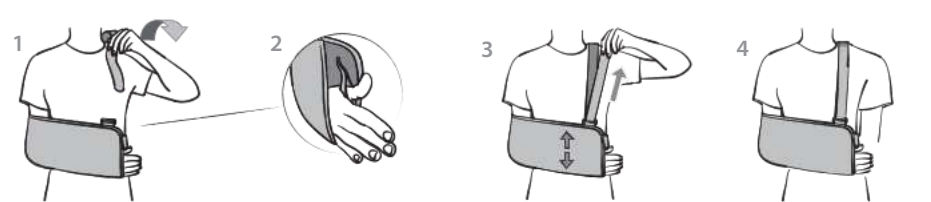
KMQ401


KSP401

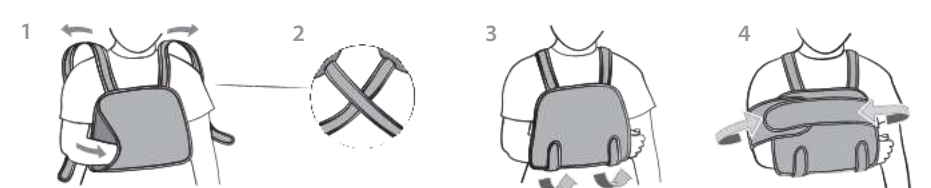KSH401

KCL401

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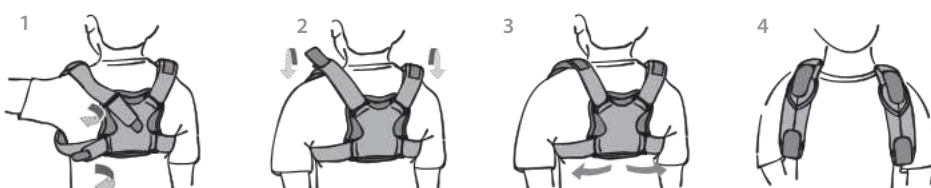EMO kids

Orthopedic Medical Specialties s.l. guarantees all its products as long as they have not been tampered with or altered and always within the framework and deadlines of the legal provisions. Products whose characteristics have been altered due to misuse are not included in this warranty.

Normativa
The medical device complies with Regulation (EU) 2017 /745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Description and characteristics

MMQ401 Wrist immobilizer: Velcro closure. It incorporates a formed and removable maleable aluminum splint. KSP401 Bag support: Extremely breathable fabric. With loop to facilitate the rest of the thumb and the position of the wrist. Protective shoulder pad. KSH401 Shoulder immobilizer: Gleno-humeral joint immobilizer support. Distributes the weight of the treated arm avoiding overload of the cervical area. Straps crossed over the back to improve support. KCL401 Clavide immobilizer: inelastic straps in "eight" of guarismo. Traction regulation by four independent adjustment points. Large pads that adapt easily and prevent chaf and overpressure. KRD401 Knee immobilizer: made of comfortable and breathable material. It incorporates three lateral and rear conformable whales. They can be adapted to achieve the desired degree of exion. Release the kneecap. Perfect adaptation to all the perimeters of the leg thanks to its two mobile side panels. Velcro dosure. KRD461 Envolping knee brace: Made with breathable and elastic material. Housing for the kneecap, avoiding pressure on it and facilitating exo-extension. Side spiral rods Elastic handles that regulate compression on the thigh and calf. Popliteal area open. Easy placement, control and review of the area. KTB461 Stabilizer ankle brace: Elastic and breathable fabric. Open on the calf for easy placement. It incorporates an elastic strap that allows the realization of a stabilizing bandage. KPC401 Cranial protector: Made with padded and breathable material. Padded chin guard. KCC401 Cervical collar: soft padding. Designed to easily adapt to the morphology of the patient. Adjustment and placement by velcro. E cts

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Efec tos
MMQ401 Wrist immobilizer: Fastening. Stabilization of the wrist joint. Controlled immobilization. KSP401 Bag support: supports the weight of the arm. Distribute the weight of the arm over the entire orthosis, including the elbow. KSH401 Shoulder immobilizer: immobilizes the shoulder joint in internal rotation. Controls exo-extension, rotation, abduction and shoulder adduction. It keeps the elbow in exion of 90º and in intermediate pronosupination. KCL401 Clavide immobilizer. It facilitates the formation of the fracture callus approaching the segments by retropulsion of the shoulders. It exerts a wide traction on the acromio-clavicular and coraco-clavicular joint. KRD401 Knee immobilizer: Immobilization and protection of the knee joint. KRD461 Envolping knee brace: Stabilization. Stimulation and improvement of proprioception. KTB461 Stabilizer ankle brace: adjustable clamping of the joint by elastic strap. Compression. Improvement of proprioception. Therapeutic heat KPC401 Cranial protector: Cranial protection. KCC401 Cervical collar: reminder to restrict head and neck movements. Support and support. It reduces the pressure on the intervertebral discs. Relief of muscle tension.

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MMQ401 Wrist immobilizer: Para-articular wrist trauma without fracture. Recurrent wrist sprain. Sequela painful or in ammatory. KSP401 Bag support: Shoulder and arm injuries. Subluxation Soft tissue injuries (burns, infections). In ammatory or painful sequelae Postoperative. KSH401 Shoulder immobilizer: Subcapital humerus fractures. Clavide fracture. Subluxation of the glenohumeral joint. Acromio-clavicular dislocation. Shoulder and arm trauma treatment. Post surgery. KRD401 Clavide immobilizer: Clavicle fracture. Postoperative. KRD401 Knee immobilizer: Trauma. Conservative treatment of kneecap injuries. Sprain and ligament tears. Pre-surgical and post-surgical immobilization. KRD461 Envolping knee brace: Knee sprain. Tendinitis. Sprain. KTB461 Stabilizer ankle brace: Sprain, Tendinitis, Sprain relapse prevention: KPC401 Cranial protector: Cerebral palsy. Epileptic patients KCC401 Cervical collar: Trauma. Cervicobrachialgia. Postoperative protec tion.

Instructions for use and placement
MMQ401: Adapt the strap to the morphology of the patient. Place the orthosis over the area at the correct height for use. Use velcro straps to close and adjust. KSP40: insert the forearm completely into the support. The elbow should reach the apex of the bag and the wrist joint should be inside the support. Pass the webbing behind the neck and adjust. The webbing can also be passed between the arm and the trunk and then over the contra-lateral shoulder. Then adjust. KSH401: insert the arm into the support. Pass the straps over the shoulders, cross them on the back and x them in the front area of the support. The additional band is then adjusted by xing it on the area that hugs the forearm, passing it around the trunk and xing its other end on the area of the orthosis that hugs the forearm. KCL401: Place the two axillary padded supports on the armpits. Pass a cinch from the end of the axillary pad over the orthosis to the back of the dorsal pad and x with velcro. Pass the other webbing from the end of the axillary pad below the armpit to the dorsal pad pin and x with velcro. Repeat the process on the other arm. Traction of each of the four adjustments symmetrically until the shoulders are retracted.

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Instrucciones de uso y colocación
MMQ401: Adapt the strap to the morphology of the patient. Place the orthosis over the area at the correct height for use. Use velcro straps to close and adjust. KSP40: insert the forearm completely into the support. The elbow should reach the apex of the bag and the wrist joint should be inside the support. Pass the webbing behind the neck and adjust. The webbing can also be passed between the arm and the trunk and then over the contra-lateral shoulder. Then adjust. KSH401: insert the arm into the support. Pass the straps over the shoulders, cross them on the back and x them in the front area of the support. The additional band is then adjusted by xing it on the area that hugs the forearm, passing it around the trunk and xing its other end on the area of the orthosis that hugs the forearm. KCL401: Place the two axillary padded supports on the armpits. Pass a cinch from the end of the axillary pad over the orthosis to the back of the dorsal pad and x with velcro. Pass the other webbing from the end of the axillary pad below the armpit to the dorsal pad pin and x with velcro. Repeat the process on the other arm. Traction of each of the four adjustments symmetrically until the shoulders are retracted.

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Lavagem e manutenção
Hand wash with warm water (maximum 30º) and neutral soap. During washing, if the garment has velcro, they must be glued together or kept closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents can damage the garment. Do not use bleach or dry clean. To dry, spread the garment and do not expose it directly to a direct heat source. Store the orthosis at room temperature, preferably in the original container.

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Instrucciones de uso e colocação
MMQ401: Adapt the strap to the morphology of the patient. Place the orthosis over the area at the correct height for use. Use velcro straps to close and adjust. KSP40: insert the forearm completely into the support. The elbow should reach the apex of the bag and the wrist joint should be inside the support. Pass the webbing behind the neck and adjust. The webbing can also be passed between the arm and the trunk and then over the contra-lateral shoulder. Then adjust. KSH401: insert the arm into the support. Pass the straps over the shoulders, cross them on the back and x them in the front area of the support. The additional band is then adjusted by xing it on the area that hugs the forearm, passing it around the trunk and xing its other end on the area of the orthosis that hugs the forearm. KCL401: Place the two axillary padded supports on the armpits. Pass a cinch from the end of the axillary pad over the orthosis to the back of the dorsal pad and x with velcro. Pass the other webbing from the end of the axillary pad below the armpit to the dorsal pad pin and x with velcro. Repeat the process on the other arm. Traction of each of the four adjustments symmetrically until the shoulders are retracted.

Instrucciones de uso e colocação
MMQ401: Adapt the strap to the morphology of the patient. Place the orthosis over the area at the correct height for use. Use velcro straps to close and adjust. KSP40: insert the forearm completely into the support. The elbow should reach the apex of the bag and the wrist joint should be inside the support. Pass the webbing behind the neck and adjust. The webbing can also be passed between the arm and the trunk and then over the contra-lateral shoulder. Then adjust. KSH401: insert the arm into the support. Pass the straps over the shoulders, cross them on the back and x them in the front area of the support. The additional band is then adjusted by xing it on the area that hugs the forearm, passing it around the trunk and xing its other end on the area of the orthosis that hugs the forearm. KCL401: Place the two axillary padded supports on the armpits. Pass a cinch from the end of the axillary pad over the orthosis to the back of the dorsal pad and x with velcro. Pass the other webbing from the end of the axillary pad below the armpit to the dorsal pad pin and x with velcro. Repeat the process on the other arm. Traction of each of the four adjustments symmetrically until the shoulders are retracted.

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Instrucciones de uso e colocação
MMQ401: Adapt the strap to the morphology of the patient. Place the orthosis over the area at the correct height for use. Use velcro straps to close and adjust. KSP40: insert the forearm completely into the support. The elbow should reach the apex of the bag and the wrist joint should be inside the support. Pass the webbing behind the neck and adjust. The webbing can also be passed between the arm and the trunk and then over the contra-lateral shoulder. Then adjust. KSH401: insert the arm into the support. Pass the straps over the shoulders, cross them on the back and x them in the front area of the support. The additional band is then adjusted by xing it on the area that hugs the forearm, passing it around the trunk and xing its other end on the area of the orthosis that hugs the forearm. KCL401: Place the two axillary padded supports on the armpits. Pass a cinch from the end of the axillary pad over the orthosis to the back of the dorsal pad and x with velcro. Pass the other webbing from the end of the axillary pad below the armpit to the dorsal pad pin and x with velcro. Repeat the process on the other arm. Traction of each of the four adjustments symmetrically until the shoulders are re