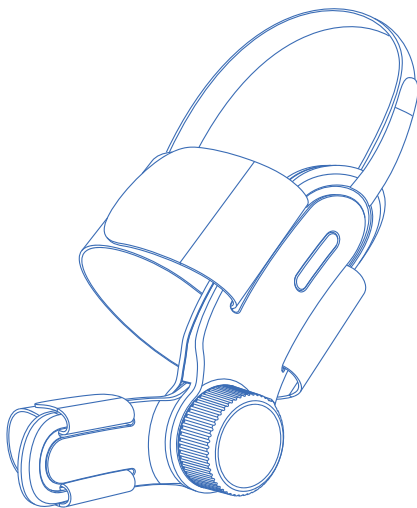
HALLUX VALGUS CORRECTOR

Thank you for your purchase of this GO Medical product.
To ensure your safety and optimal performance, please read the
following instructions thoroughly before use.

DESCRIPTION OF IDENTIFICATION

► EN

	CE Marking		UKCA Marking
	Medical Device		Manufacturer
	Recycling		The Green Dot
	Consult instructions for use or consult electronic instructions for use		

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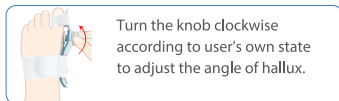
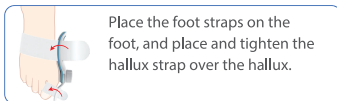
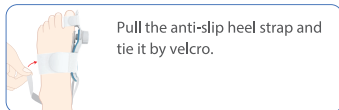
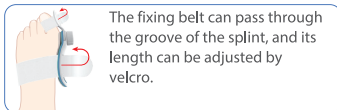
How to Use Product

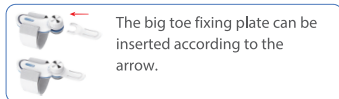
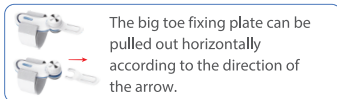
GO Medical Hallux Valgus Corrector

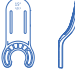
Strap

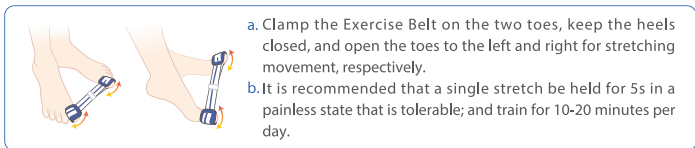
Fixing Plate

Tips for the Use of Toe Fixing Plate

Toe Valgus Angle	Period	Recommended
0°-15° 	Reversible	0° Fixing Plate. 
15°-25° 	Contracture	7.5° Fixing Plate; Use other angle fixing plates after adaptation. 
25°-40° 	Severe	15° Fixing Plate; Use other angle fixing plates after adaptation. 

Exercise Belt

▶ **Intended Use**

Correct the angle of the big toe bone deformity and relief pressure on the big toe, relieve and recover the pain and deformity symptoms gradually, and prevent further degeneration.

▶ **Technical Parameters**

Knobs for distance adjustment: 0-8mm

Angle specification of big toe fixing plate: 0°, 7.5°, 15°

▶ **Prohibition**

1. Read Before Use

- 1) Please check the raw materials if you suffer from allergic diseases.
- 2) If you feel pain, swelling, sensation changes or abnormal reversal occur during use, please stop using and seek medical attention immediately.
- 3) Contraindicated for those with bleeding tendency.
- 4) Those with abnormal skin and wound ulcers should avoid using it in abnormal areas and wounds areas.

2. The product should be carefully used on below symptom

- Gout
- Arthritis
- Rheumatic disease
- Polyneuropathy and soft tissue swelling

3. Important Warning

If you feel pain, swelling, sensation changes or abnormal reversal occur during use, please stop using and seek medical attention immediately.

4. Cautions

- Ensure that the toe joints and the splint are aligned so that the range of motion is not restricted.
- Can only be worn for about 1 hour for the first time. After you getting used to it, it can be worn while sleeping or at home.

5. Additional Notes

- Please wear shoes with elasticity and wide toe caps for faster results.
- Before use please check for damage or deformation. If there is any abnormality, please don't use it.

▶ **Washing Method**

- Wash with water or soap, at a temperature lower than 40° C / 104° F and hang to dry.
- Do not use bleach, softener, iron, dryer or dry cleaning.

▶ Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

▶ Transport

During transportation, avoid splashing, inversion and collision from rain and snow;

Please store in a well-ventilated place;

Keep the product away from acids, bases, fats, oils;

Keep the product away from heat source and sunlight.

▶ Storage Method

Environment temperature: -20°C ~ +50°C

Relative humidity: ≤90%

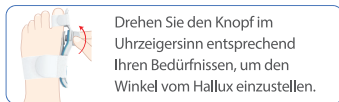
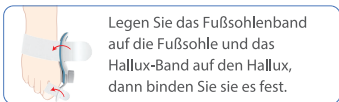
This product should be stored in a dry and well-ventilated room. Don't mix it with corrosive substances such as acid and alkali. Keep it away from fire.

▶ Specification

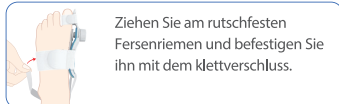
Product	GO Medical Hallux Valgus Corrector	
Model	GM-HVC	
Specification	<ul style="list-style-type: none"> - Bunion Corrector x 1 (with 0° Fixing Plate and Heel Strap) - 7.5° Fixing Plate x 1 - 15° Fixing Plate x 1 - Plus Size Strap Set x 1 - Exercise Band x 1 - Storage Bag x 1 	
Main Material	Splint: Gel Insole: Fixing Strap:	PC, POM Nylon, PU Nylon
Product Size	155×40×33mm ±10%	
Net Weight	62g±10%	
Shelf Life	5 Years	
Lot Number	See Package	

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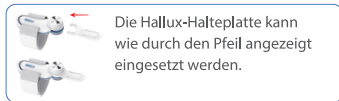
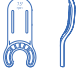
GO Medical Hallux Valgus Corrector

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